

atemious^{PRO}



MADE IN
GERMANY

COMFORTABLE UNIVERSAL FLEECE MASK

The fold-able universal mask Atemious Pro offers effective protection against infection and is extremely comfortable to wear. The mask meets the requirements as a corona pandemic respirator.



KIND TO SKIN

COMFORTABLE BREATHING

SKIN FRIENDLY

ATEMIOUS PRO
ART. 2001

According to the test principle for Corona COVID-19 Pandemic Respirators

<https://www.airflow.com/facemasks>


Order using the quick order pad on our homepage using the part number 10000945.

DIRECTIONS FOR USE

When using the mask you must read and follow the following information. Failure to follow the instructions for use can lead to illness or death.

- The user of the mask must be trained to use it correctly
 - Only for use for infection protection
- Atemious Pro** masks do not protect against gases and vapors.

- The oxygen content in the breathing air must be minimum 19.5% vol.
- Respiratory protection must not be used if the concentration, nature and property of the hazards are not known.
- The mas has to be changed immediately if it is damaged or the breathing window level increases. After the end of a work shift the mask must be changed in principle.
- Never make changes to the mask.

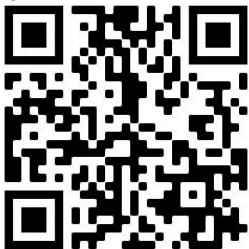

UNIVENT⁺
MEDICAL

INSTRUCTIONS



1. Wash hands before touching the mask. Take the respirator out of the packaging, unfold it and pull the ear band firmly in both directions. Inspect the mask visually for damage.



2. Place the respirator in your hand with the nose area facing up. Now direct the respirator to your face first insert your chin, then pull the mask over your nose. Fix the side of the ear bands.



3. For a firm fit, it is necessary to adjust the clip in the nose area to contour the face. Warning: hair like beards can affect the required firm fit of the mask and can cause leaks .



4. Checking the tightness - cover the mask with both hands and exhale vigorously. With slight pressure exhale in the mask, no air should escape the edges. When used correctly, the filter material moves slightly when inhaling.

STORAGE AND SHELF LIFE

Shelf life when properly stored max. 3 years. Check expiry date on masks before use. Storage and transport clean and dry between -5°C to + 25°C at max. 60% humidity in original packaging. Protect against UV radiation!

DISPOSAL

In the event of damage, high pollution or if the breathing resistance is significantly increased, the mask should be changed and disposed in accordance with regional regulations.

CONFORMITY ASSESSMENT

The particle filtering half mask meets the requirements of the European Directive (EU) 2016/425. Type examination and conformity assessment procedures were carried out with the participation of the following notified body: Universal Certification and Surveillance Service Trade Ltd. CoYukarı Dudullu Mahallesi, KEYAP E2 No:84, 34775 Dudullu Organize Sanayi Bölgesi/Ümraniye/Istanbul, Turkey

Declaration of conformity: www.univent-medical.de/conformity

INFO

We are happy to help you choosing the right respirator. The selection of the most suitable personal protective equipment (PPE) depends on the respective situation and may only be carried out by competent persons. For questions and further information. Please contact Airflow Developments.

<https://www.airflow.com/facemasks>